

Convergys®HypocleanCC 100 ml

REF
1100-1704
LOT
8010350114

2015-07
+15°C  **+30°C**


Convergent Technologies GmbH & Co. KG
Ringstraße 14
D-35091 Coelbe - Germany


IVD

Warning

For/pour/für/por:
Convergys®X5

Cleaning and bleaching concentrate for blood cell counters.

Solution concentrée de nettoyage et de lavage pour les compteurs de cellules sanguines.

Konzentrierte Reinigungs- und Bleichlösung für Blutanalysegeräte.

Solución concentrada de limpieza con lejía para los contadores hematológicos.

Ingredients:
Detergents < 1.0 %
Sodium hypochlorite < 5.0 %
Sodium hydroxide < 0.5 %
Stabilizers < 0.5 %

H315 Causes skin irritation
H319 Causes serious eye irritation.
P305 + P351 + P338

www.convergent-technologies.de

CTX5H_20140417Ver003

1. Intended Use:

Alkaline hypochlorite solution for intensive oxidative cleaning of hematology analyzers' capillaries, tubing and chambers, removing blood component precipitates and lipoprotein deposits.

2. Composition/Data on components:

Name	Concentration	Symbol	Risk
Sodium hypochlorite, solution	< 15.0 %	C	R31-34
Sodium hydroxyde	< 3.0 %	C	R35

3. Warning:

Hazards identification

Most important hazards:

Xi Irritant

Information pertaining to particular dangers for man and environment:

R36/38, Irritating to eyes and skin.

Classification system:

The classification is conform to current EC-directives.

Acute toxicity:

Sodium hypochlorite: LD50: 4445 mg/kg (oral, rat), 5800 mg/kg (oral, mouse).

Primary irritant effect:

on the skin: redness

on the eye: redness

Sensitization:

No sensitizing effect known.

Convergent Technologies GmbH & Co. KG

Registered Office

Ringstrasse 14, 35091 Cölbe, Germany
Tel: +49 6421 8869948 Fax: +49 6421 8869958

Homepage: www.convergent-technologies.de

e-mail: info@convergent-technologies.de

Ust-Id: DE 241291597

Steuernummer: 031 310 30121

Amtsgericht Marburg HRA 4125

Geschäftsführer: Umesh Bhalla

Bank: Volksbank Main-Tauber eG, Tauberbischofsheim, Germany

EURO Transfers

Account No. 67131401
IBAN DE66 6739 0000 0067 1314 01
BIC GENODE61WTH
SWIFT GENODEFF

USD Transfers

Account No. 567131408
IBAN DE97 6739 0000 0567 1314 08
BIC GENODE61WTH
SWIFT GENODEFF

Form: RI1601 REV002

Additional toxicological information:

Irritant

4. Precautions and storage:

Convergys[®] Hypoclean CC solution is design for professional laboratory use. The usual laboratory precautions should be observed. *Convergys*[®] Hypoclean CC solution is to be shipped and stored dry at temperatures between 18 – 30° C.

5. Stability:

Unopened bottles are stable through the expiration date. Opened bottles are stable for 120 days.

6. Indications of Deterioration:

If any solid particle or contaminated particle is seen in the container it may indicate deterioration. **Do not use the product if it deterioration is suspected.**

7. Instructions:

Convergys[®] Hypoclean CC solution is to be installed by professional user on *Convergys*[®] X5 automated Hematology Analyzers as per instructions in Analyzers User Manual.

1. Open the *Convergys*[®] Hypoclean CC bottle.
2. Pour the required amount of *Convergys*[®] Hypoclean CC in a tube.
3. Put the tube in the Sample adapter as applicable.

8. Limitations:

The performance of this product is assured only if it is properly stored and used as described in this insert.

9. Technical Assistance: Tech. support and Customer service at at <http://www.convergent-technologies.de/support/>**10. Manufactured by:** Convergent Technologies GmbH & Co. KG, Ringstrasse 14, 35091 Coelbe, Germany

Convergent Technologies GmbH & Co. KG**Registered Office**

Ringstrasse 14, 35091 Cölbe, Germany
Tel: +49 6421 8869948 Fax: +49 6421 8869958

Homepage: www.convergent-technologies.de

e-mail: info@convergent-technologies.de

Ust-Id: DE 241291597

Steuernummer: 031 310 30121

Amtsgericht Marburg HRA 4125

Geschäftsführer: Umesh Bhalla

Bank: Volksbank Main-Tauber eG, Tauberbischofsheim, Germany

EURO Transfers

Account No. 67131401
IBAN DE66 6739 0000 0067 1314 01
BIC GENODE61WTH
SWIFT GENODEFF

USD Transfers

Account No. 567131408
IBAN DE97 6739 0000 0567 1314 08
BIC GENODE61WTH
SWIFT GENODEFF

Form: RI1601 REV002