

Convergys® Lyse-5P

Lysing Agent

Code: 1100-1702

REF 1100-1702

Convergys® Lyse-5P

5 L

LOT 3010000111

for/pour/für/para:
Convergys® X5

Ingredients:



2013-01



+15°C
+30°C



Convergent Technologies GmbH & Co. KG
Ringstraße 14
D-35091 Coelbe - Germany



Surfactants < 3.6%
Buffers < 1.0%
Preservative < 0.4%
Stabilizers < 0.4%
in ion-free water.

On Board Stability: 120 days.
For in-vitro diagnostic use only.



www.convergent-technologies.de

CTX5L5_20131312Ver002

1. Intended Use:

Lysing agent for stromatolysis of erythrocytes (RBC), for quantitative determination of leukocytes (WBC), leukocyte five-part differentiation (LYM, MON, NEU, EOS, BAS) and hemoglobin (HGB) concentration measurement in human blood on automated hematology analyzer.

2. Composition/Data on components:

CAS No.: 7732-18-5 water
EINECS Number: 231-791-2

3. Warning:

Hazards designation:

Void

Specific risks:

None determined.

Information pertaining to particular dangers for man and environment:

The product does not have to be labeled due to the calculation procedure of the "General Classification guideline for preparations of the EU" in the latest valid version.

Classification System:

The classification is conform to current EC-directives

Toxicological information

Acute toxicity:

When used and handled according to specifications, the product does not have any harmful effects according to our experience and the information provided to us.

The material is not subject to classification according to EC lists in the last version.

Convergent Technologies GmbH & Co. KG

Registered Office

Ringstrasse 14, 35091 Cölbe, Germany
Tel: +49 6421 8869948 Fax: +49 6421 8869958
Homepage: www.convergent-technologies.de
e-mail: info@convergent-technologies.de

Ust-Id: DE 241291597
Steuernummer: 031 310 30121
Amtsgericht Marburg HRA 4125
Geschäftsführer: Umesh Bhalla

Bank: Volksbank Main-Tauber eG, Tauberbischofsheim, Germany

EURO Transfers

Account No. 67131401
IBAN DE66 6739 0000 0067 1314 01
BIC GENODE61WTH
SWIFT GENODEFF

USD Transfers

Account No. 567131408
IBAN DE97 6739 0000 0567 1314 08
BIC GENODE61WTH
SWIFT GENODEFF

Form: RI1601 REV002

Sensitization:

No sensitizing effect known.

Chronic toxicity:

Not determined.

Inhalation:

Not determined.

Primary irritant effect:

on the skin: No irritant effect.

on the eye: redness.

Ingestion:

Not determined.

4. Precautions and storage:

Convergys[®] Lyse-5P solution is design for professional laboratory use. The usual laboratory precautions should be observed. *Convergys*[®] Lyse-5P solution is to be shipped and stored dry at temperatures between 18 – 30° C.

5. Stability:

Unopened bottles are stable through the expiration date. Opened bottles are stable for 120 days.

6. Indications of Deterioration:

This reagent is colorless. If any solid particle or contaminated particle is seen in the container it may indicate deterioration. **Do not use the product if it deterioration is suspected.**

7. Instructions:

Convergys[®] Lyse-5P solution is to be installed by professional user on *Convergys*[®] X5 automated Hematology Analyzers as per instructions in Analyzers User Manual.

1. Open the cap of the bottle
2. Connect the cap in which the there is a tubing connection.
3. Connect the tube to the cap of the Reagent on one side and to the back of the instrument in the Lyse-5P inlet on the other side

8. Limitations:

The performance of this product is assured only if it is properly stored and used as described in this insert.

9. Technical Assistance: Tech. support and Customer service at at <http://www.convergent-technologies.de/support/>**10. Manufactured by:** Convergent Technologies GmbH & Co. KG, Ringstrasse 14, 35091 Coelbe, Germany

Convergent Technologies GmbH & Co. KG**Registered Office**

Ringstrasse 14, 35091 Cölbe, Germany

Tel: +49 6421 8869948 Fax: +49 6421 8869958

Homepage: www.convergent-technologies.de

e-mail: info@convergent-technologies.de

Ust-Id: DE 241291597

Steuernummer: 031 310 30121

Amtsgericht Marburg HRA 4125

Geschäftsführer: Umesh Bhalla

Bank: Volksbank Main-Tauber eG, Tauberbischofsheim, Germany

EURO Transfers

Account No. 67131401

IBAN DE66 6739 0000 0067 1314 01

BIC GENODE61WTH

SWIFT GENODEFF

USD Transfers

Account No. 567131408

IBAN DE97 6739 0000 0567 1314 08

BIC GENODE61WTH

SWIFT GENODEFF

Form: RI1601 REV002